



## Body Fluids Calibration Verification / Linearity Test Kit

### INTENDED USE

**VALIDATE BODY FLUIDS** Calibration Verification / Linearity Test Kit solutions are intended for in vitro diagnostic use in the determination of linearity, calibration verification and verification of reportable range on automated instrument systems for the following analytes: Set 1: albumin (ALB), amylase (AMY), cholesterol (CHOL), creatinine (CREA), glucose (GLU), lactate (LAC), lactate dehydrogenase (LD), total protein (TP), triglycerides (TRIG) and urea nitrogen (UN). Set 2: CSF Total Protein (CSF-TP).

Each test kit consists of two sets of one bottle each of Levels 1 through 5. Set 1 contains 4.0 milliliters. Set 2 contains 2.0 milliliters. For each set, there exists a linear relationship among Levels 1 through 5.

### SUMMARY

For each **VALIDATE** Calibration Verification / Linearity Test Kit, multiple levels are provided to establish the relationship between theoretical and actual performance of each of the included analytes. The **VALIDATE** Calibration Verification / Linearity Test Kit will assist in the documentation of linearity, calibration verification and verification of linear range required by many inspection agencies. The solutions will also provide assistance when troubleshooting instrument systems, reagent problems and calibration anomalies.

### REAGENTS

#### Reactive Ingredients:

Purified chemicals and constituents of human and/or animal source in a synthetic matrix designed to simulate cerebrospinal, peritoneal ascites, pleural and pericardial fluids<sup>1,2,3,4,5</sup>.

Note: Some raw materials are purified from porcine.

#### Nonreactive Ingredients:

Preservatives and stabilizers.

#### Precautions and Warnings:

*For In Vitro Diagnostic Use*

Disposal of all waste material should be in accordance with local guidelines.

#### WARNING: Potentially Biohazardous

Human source material is considered potentially biohazardous. Material of human origin used in the manufacture of this test kit was tested at the donor level using FDA or CE approved methods and found to be non-reactive for HBV, HCV and HIV. Because no test method can offer complete assurance that infectious agents are absent, these specimens should be handled and treated as potentially infectious.

### STORAGE AND STABILITY

The **VALIDATE BODY FLUIDS** Calibration Verification / Linearity Test Kit is stored at -10° to -25°C. **Do NOT store in a frost-free freezer.** Test kits are stable until the expiration date printed on the bottle and storage container when handled according to instructions. **A maximum of four (4) freeze-thaw cycles is recommended.**

### PREPARATION

Prior to use, remove the **VALIDATE BODY FLUIDS** Calibration Verification / Linearity Test Kit from storage and allow to come to room temperature (18° to 25°C). Invert gently several times before dispensing.

To maximize stability, it is recommended that exposure to room temperature be minimized. Tightly cap opened bottles and return to -10° to -25°C immediately after dispensing.

Discard any solutions that appear to have gross bacterial contamination.

**VALIDATE** Calibration Verification / Linearity Test Kits should be treated in the same manner as patient samples. If dilutions or pre-treatment are required as part of the testing procedure, follow the manufacturer's instructions.

### ASSAY

Analyze each level in replicates. If following the CLSI EP6 guidelines for linearity, use a random analytical sequence to assay each level.

### CALCULATION OF RESULTS

**VALIDATE** Calibration Verification / Linearity material is prepared in a manner such that an equal distance (delta) exists between Levels 1 through 5. This dilution scheme is consistent with the CLSI EP6 recommendation for preparing linearity sets.

Two examples for calculating the theoretical values of Levels 1 through 5 are provided below.

#### Example 1:

Choose two consecutive levels and calculate the delta between the recovered values. The following example demonstrates the use of the delta between Levels 2 and 3 to calculate the theoretical value for Levels 1, 4, and 5.

Level 3 – Level 2 = Delta

Level 1 Theoretical = Level 2 Recovered – Delta

Level 4 Theoretical = Level 3 Recovered + Delta

Level 5 Theoretical = Level 4 Theoretical + Delta

NOTE: The user can select the calculated delta between any two consecutive levels to calculate the theoretical values. Typically, the user should choose an area of recovery known to be linear for the method being studied.

#### Example 2:

Theoretical values can be determined using the recovered values for Levels 1 and 5. Using this method, the following formulas apply:

Level 2 Theoretical = 0.75 \* (Level 1) + 0.25 \* (Level 5)

Level 3 Theoretical = 0.5 \* (Level 1) + 0.5 \* (Level 5)

Level 4 Theoretical = 0.25 \* (Level 1) + 0.75 \* (Level 5)

After theoretical values are calculated, for each analyte plot the expected (Theoretical) value on the x-axis versus the Recovered value on the y-axis using standard linear graph paper. If the system is linear, the plot should approximate a straight line. The point at which the line is no longer straight can be used to determine the limit of linearity or the reportable range.

Data reduction is available from LGC Maine Standards. Commercially available linear regression software may also be used. The software should provide data point display and x-y graphical presentation. Linear regression should be interpreted using standard statistical analysis and the results should be compared with the instrument manufacturer's claims for linearity or with individual laboratory performance requirements. The degree of acceptable nonlinearity is an individual judgment based on methodology, clinical significance and medical decision levels of the test analyte.

### LIMITATIONS

**VALIDATE** Calibration Verification / Linearity Test Kit solutions are not intended for use as routine quality control materials or as calibration materials.

### EXPECTED VALUES

**VALIDATE** Calibration Verification / Linearity Test Kits are manufactured such that an equal distance (delta) exists between levels as recommended by CLSI EP6 for assessing linearity. As the distance between levels is equal, any two levels can be held to be 'true' when assayed and the theoretical values for each of the other three levels can be calculated allowing this test kit to be used on multiple automated instrument systems.

## TYPICAL VALUES

Actual results obtained may vary depending on instrumentation, methodology and assay temperature. Results may also be dependent on the accuracy of the instrument / reagent system calibration. The degree of acceptable nonlinearity is an individual judgment based on methodology, clinical significance and medical decision levels of the test analyte.

Typical recovered values for Level 1 and Level 5 are presented in the table(s) provided. Typical values for mid-levels are based on an equal distance (delta) between levels.

Typical Recovered Values on Roche COBAS®						
205bf Lot #: 25AK089210						
Analyte	Units	Level 1	Level 2	Level 3	Level 4	Level 5
ALB	g/dL	0.4	1.7	3.1	4.4	5.7
AMY	U/L	7	297	586	876	1165
CHOL	mg/dL	8	189	369	550	730
CREAT	mg/dL	0.4	5.6	10.9	16.1	21.3
GLU	mg/dL	5	180	356	531	706
LAC	mg/dL	4	36	68	99	131
LD	U/L	15	222	430	637	844
TP	g/dL	0.6	3.3	5.9	8.6	11.2
TRIG	mg/dL	17	222	427	631	836
UN	mg/dL	4	28	51	75	98
CSF-TP	mg/dL	5	52	100	147	194

205bf Lot #: 25AK089210						
Analyte	SI Units	Level 1	Level 2	Level 3	Level 4	Level 5
ALB	g/L	4	17	31	44	57
AMY	U/L	7	297	586	876	1165
CHOL	mmol/L	0.207	4.882	9.557	14.232	18.907
CREAT	μmol/L	35.2	495.5	955.9	1416.2	1876.5
GLU	mmol/L	0.278	10.004	19.731	29.457	39.183
LAC	mmol/L	0.444	3.968	7.493	11.017	14.541
LD	U/L	15	222	430	637	844
TP	g/L	6	33	59	86	112
TRIG	mmol/L	0.192	2.506	4.820	7.133	9.447
UN	mmol/L	1.428	9.818	18.207	26.597	34.986
CSF-TP	g/L	0.1	0.6	1.0	1.5	1.9

- Ohtsuki, C., Ph.D. (n.d.). How to prepare the simulated body fluid (SBF) and its related solutions, Retrieved March 20, 2017, from <http://mswebs.naist.jp/LABs/tanihara/ohtsuki/SBF/>.
- Deepa C, Muralidhar K. Renal replacement therapy in ICU. J Anaesthesiol Clin Pharmacol [serial online] 2012 [cited 2017 Mar 20];28:386-96. Available from: <http://www.joacp.org/text.asp?2012/28/3/386/98357>.
- Gibson, A. T., & Segal, M. B. (1978). A study of the composition of pericardial fluid, with special reference to the probable mechanism of fluid formation. The Journal of Physiology, 277, 367–377.
- Hladky, S. B., & Barrand, M. A. (2014). Mechanisms of fluid movement into, through and out of the brain: evaluation of the evidence. Fluids and Barriers of the CNS, 11, 26. <http://doi.org/10.1186/2045-8118-11-26>.
- Meyers, D. G., Meyers, R. E., and Prendergast, T.W. (1997). The Usefulness of Diagnostic Tests on Pericardial Fluid. The Chest Journal. 111(5), 1213-1221. DOI: <http://dx.doi.org/10.1378/chest.111.5.1213>.

## ORDERING INFORMATION

ORDER NO.: 205bf

### VALIDATE BODY FLUIDS

Calibration Verification / Linearity Test Kit:

Set 1: 5 x 4 mL

Set 2: 5 x 2 mL



**CE Symbols** – This product fulfills the requirements of the European Directive 98/79/EC for *in vitro* medical devices. The following symbols may be used where applicable in labeling for Maine Standards Company products:

	Lot Number
	Expiration Date
	Manufacturer
	Storage Temperature
	In Vitro Diagnostic Medical Device
	Catalog Number
	Insert
	Biological Risk
	Wellkang Ltd (www.CE-marking.eu) 29 Harley St., London W1G 9QR, UK

For a list of countries in which VALIDATE® is registered see: [http://www.mainestandards.com/Products/ce\\_reg.php](http://www.mainestandards.com/Products/ce_reg.php)

## Rx Only

## CONTACT INFORMATION:

Maine Standards Company, LLC.  
221 US Route 1  
Cumberland Foreside, ME 04110 USA

1-800-377-9684  
+1-207-892-1300  
+1-207-892-2266 Fax  
[msc.sales@LGCgroup.com](mailto:msc.sales@LGCgroup.com)  
[msc.techsupport@LGCgroup.com](mailto:msc.techsupport@LGCgroup.com)  
[www.mainestandards.com](http://www.mainestandards.com)